## Medical use scar therapy gel – Gel zur medizinischen Narbenbehandlung – Gel à usage médical pour le traitement de cicatrices – Gel per uso medico per il trattamento delle cicatrici

### [EN] Consumer Information

A flexible wound dressing for abnormal scar prevention, and treatment of open wounds and compromised skin.

When used as directed Stratamed dries to form a protective layer that is gas permeable and waterproof which hydrates and protects compromised skin areas and open wounds from chemical and microbial invasion.

Stratamed can be applied to fresh incisions and excisions, open wounds and compromised skin surfaces including those from post laser treatments, chemical peels, dermal abrasion, 1<sup>st</sup> and 2<sup>nd</sup> degree burns and superficial radiation therapy burns.

#### For open wounds and compromised skin:

Stratamed dries to form an inert, thin, flexible wound dressing which hydrates and protects affected areas. Stratamed helps create an optimal wound healing environment for superficial wounds and burns, which leads to faster reepithelialization and a reduced inflammatory response.

#### For abnormal scar prevention and treatment:

Stratamed is a self-drying, non-sticky, transparent, silicone gel formulation for the prevention and treatment of abnormal scars in the form of hypertrophic scars and keloids. Stratamed softens and flattens raised scars and allows scars to normalize the collagen synthesis process, producing a normal mature scar.

Stratamed can be used with or without a secondary protective dressing. Stratamed is ideal for exposed areas like the face and neck as well as joints and hairy areas without the need for shaving. Stratamed is used to relieve, itching and discomfort and reduce redness and discoloration associated with damaged skin and scars.

Stratamed was developed for use on all types of superficial wounds, burns, skin tears and abrasions including those resulting from:

- General Surgery
- Trauma
- Burns
- Radiation therapy dermatitis and burns
- Bites
- Grazes
- Laser therapy, including tattoo removal
- Chemical peels
- Dermal abrasion
- Acne and other diseases of the skin

#### Intended use

Stratamed may be applied to open wounds, burns and areas where skin integrity has been compromised.

#### Stratamed is used for:

- The protection and hydration of superficial wounds, burns and compromised skin.
- The prevention and treatment of keloids and hypertrophic scars.

Stratamed may also be used in conjunction with invasive skin management options to improve overall results, including surgical excision and resuturing, intralesional steroid injections, laser treatments and pressure garments.

Stratamed is suitable for children or people with sensitive skin. Stratamed may be used on infants, during pregnancy and while breastfeeding.

#### Directions for use

- Ensure that the affected skin or wound is clean.
- Pat dry as much excess exudates or wound fluid from the wound as possible prior to gel application.

#### On damaged or broken skin not requiring a secondary dressing:

Apply a very thin layer of Stratamed to the affected area and allow the gel to dry. Stratamed should be applied once daily, or twice daily to exposed areas or as advised by your physician. Once dry, Stratamed may be covered by sun screen, cosmetics, pressure garments or casts.

#### On damaged skin or wounds requiring a secondary dressing:

Apply a thin layer of Stratamed then cover with the secondary dressing. Stratamed should be reapplied when changing the dressing or checking wound progress, or as advised by your physician.

For best results Stratamed should be maintained in continuous contact with the skin (24 hrs/day).

When applied correctly to exposed areas, Stratamed should be dry in 5–6 minutes. If it takes longer to dry you have probably applied too much. Gently remove the excess with a clean tissue or gauze and allow the drying process to continue.

#### Recommended duration of treatment

Stratamed is recommended for a minimum treatment of 60–90 days (24/7). Larger scars may take longer. Continued use is recommended until no further improvement is seen.

#### How much Stratamed do I need?

- **Size 5 gm (0.17 oz)** is enough to treat a 2–4 cm (1–2 inch) linear incision twice per day for over 90 days.
- **Size 10 gm (0.35 oz)** is enough to treat a 5–8 cm (2–3 inch) linear incision twice per day for over 90 days.
- **Size 20 gm (0.7 oz)** is enough to treat a 10–12 cm (4–5 inch) linear incision twice per day for over 90 days or an area of 6 × 12 cm (2.5 × 5 inch) twice per day for over 60 days.
- **Size 50 gm (1.75 oz)** is enough to treat an area of 12 × 15 cm, (5 × 6 inch) twice per day for over 60 days.

#### Caution

- Stratamed should not be placed in contact with the eyes.
- Stratamed should not be applied over other skin treatments without the advice of your physician.
- Stratamed may stain clothing if not completely dry.
- Stratamed does not contain anti-infective ingredients.
- Should your wound show signs of infection or fail to heal, consult your physician.
- For external use only.
- If irritation occurs, discontinue use and consult your physician.
- Not suitable for wet, highly exudative, tunnelling wounds or 3<sup>rd</sup> degree burns
- Keep out of the reach of children.
- Store below 25 °C (77 °F).
- Do not use after the expiration (EXP) date printed on the tube.

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Stratapharma AG, CH-4051 Basel